

POLICY TITLE: HYFRECTOR

POLICY STATEMENT: Health Center will have a hyfrecator available during Procedures to maintain hemostasis.

PROCEDURE:

A. Purpose:

1. To limit the amount of blood loss.

B. Equipment

1. Hyfrecator
2. Hyfrecator tips

C. Steps

1. Plug the hyfrecator into an electrical socket.
2. Place the hyfrecator tip in the handle.
3. Turn the switch to the on position.
4. Adjust the cautery level using the "up, down" buttons to achieve the wattage desired by the practitioner
5. Assist as needed.